

Stat-A-Dyne Elbow Lantz Medical

Fitting Instructions




For video instructions on how to use your Stat-A-Dyne device, scan the QR Code above.


Questions? Feel free to call us at 866-236-8889, you can also find more information on all of our products at www.lantzmedical.com.

*It is recommended to be seated or lying down while using your Stat-A-Dyne contracture management device.

- **Start with the device in a neutral position that is within your comfortable ROM.**
- **Slide your arm into the device. (The hinge of the device should line up with the elbow joint.) The hinge should be on the lateral (outside) side of the arm.**

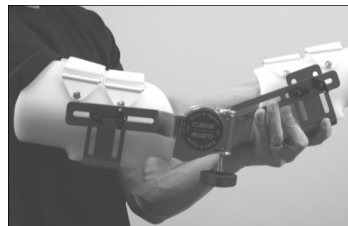

- **Adjust the position of the forearm and humeral cuffs.**
 1. The length of the brace should be telescoped so the forearm cuff fits flush with the patient's hand. The hinge of the device should be centered at the elbow joint.
 2. There should be 3 finger widths between the top of the humeral cuff and the crease of your axilla.
 3. After the length is set, you may tilt, elevate and depress your cuffs via the "L" brackets to ensure an optimal fit. (The frame of the device should line up with the long bones of the arm.)

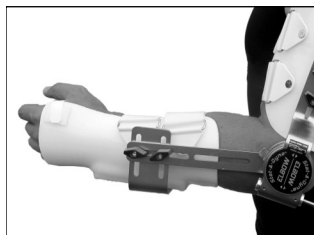

- **Once the arm is properly aligned, tighten the forearm and humeral straps.**
- **You are now ready to begin your treatment session.**