



ThuasneUSA / Lantz Medical
*Phone: 866-236-8889
*Fax: 877-406-4872
* Web: www.lantzmedical.com

PIP Extension Splint Fitting Instructions

Questions? Feel free to call us at **866-236-8889**, you can also find more information on all of our products at www.lantzmedical.com.

***It is recommended to be seated while utilizing the device.**

APPLYING THE LANTZ MEDICAL PIP EXTENSION SPLINT:

- Loosen the counterforce strap.
- Flex the splint by pinching splint together to release tension. If necessary, loosen screws.
- Slide finger into splint so that the cams are in line with the finger's PIP joint. Place finger on top of finger cradle. See Figure 1
- Secure counterforce strap over the top of finger.
- Release flexed position of splint to allow extension of the finger.
- While wearing the Lantz Medical splint it is best to be relaxed and seated with the hand level with your heart or while sleeping with your hand laying on a pillow above your heart.

REMOVING THE LANTZ MEDICAL PIP EXTENSION SPLINT:

- Loosen the counterforce strap.
- Flex splint by pinching splint together to release tension. If necessary, loosen screws.
- Remove finger from splint.

Figure 1.

