



ThuasneUSA / Lantz Medical

*Phone: 866-236-8889

*Fax: 877-406-4872

* Web: www.lantzmedical.com


Stat-A-Dyne Pro/Sup

Fitting Instructions

Questions? Feel free to call us at **866-236-8889**, you can also find more information on all of our products at www.lantzmedical.com.

***It is recommended to be seated or lying down while using your Stat-A-Dyne contracture management device.**

- **Telescope the device to the appropriate length for your forearm. (Adjust the length by loosening the two screws on the bottom of the device.)**
- **Once the device's length has been adjusted, apply the device by first putting your forearm into the distal (far) cuff and inserting your thumb through the thumb hole. *The thumb should fit comfortably through the thumb hole to avoid any pressure on the superficial branch of the radial nerve.**

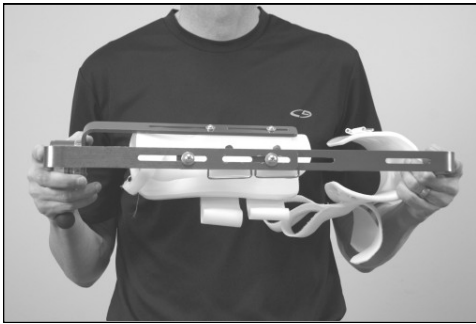

- **After your forearm is in the device, you may then pull the humeral cuff over the bicep/tricep. It is ideal to adjust the height of the humeral cuff so there are three finger widths between the top of the cuff and the crease of your axilla.**



- **Tighten the forearm and humeral straps.**
- **You are now ready to begin your treatment session.**